



**GoTalk<sup>®</sup>  
Select**



**GoTalk<sup>®</sup>Go**



- Portable
- Excellent sound quality
- Rechargeable

## **GoTalk® Wearables!**

Record and play messages on these wearable, rechargeable AAC devices from Attainment Company.

The **GoTalk® Select** has four message keys, each with three levels. Wear the GoTalk Select with a lanyard or slide-on clip. Magnets on the back attach to any metallic surface.

The **GoTalk® Go** is worn like a watch and features three message keys, each with three levels. Overlays are protected by a removable plastic cover.

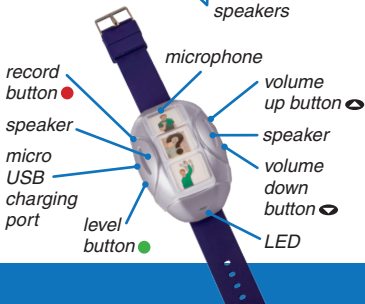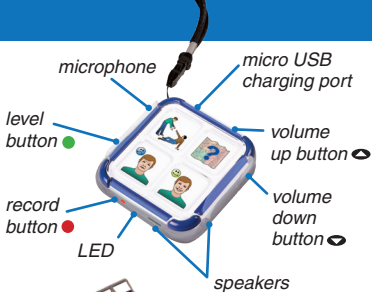

## Instructions for Use

Plug in the device to charge for 6 hours before initial use.

Refer to the **GoTalk<sup>®</sup> Go** and **GoTalk<sup>®</sup> Select** layout diagrams.

**To turn on:** Press and hold the **Volume Up button** until tone is heard.

**To turn off:** Press and hold the **Volume Down button** until tone is heard.

**Record mode on/off:** Press and release red **Record button** with a fingernail or closed pen. When in Record mode, a red light is on.

**To record a message:** Touch a **message key**. A blinking, red light indicates recording is in progress. Record for up to 28 seconds. Touch any key to stop recording.

*NOTE: To check the recorded message without leaving Record Mode, press either **Volume button**.*

**To play message:** With Record mode off (no red light), touch a **message key** to play the message. While playing messages, **Volume buttons** adjust volume up and down.



# Attainment Company

P.O. Box 930160  
Verona, WI 53593 USA  
Phone 1-800-327-4269  
International Calls  
+1-608-845-7880

**[www.AttainmentCompany.com](http://www.AttainmentCompany.com)**

© 2020 Attainment Company, Inc.

All rights reserved.

**To change levels:** Press green **Level button** with a fingernail or closed pen. Level 1 beeps once with green light. Level 2 beeps twice with blue light. Level 3 is three beeps with purple light.

*NOTE: When changing levels in Record mode, the correct number of beeps is heard for each level, but the light remains red.*

**To erase all messages:**  
Press and hold the **Record button** for 10 seconds until tone is heard.

**To insert overlays:** Remove the protective plastic cover. Add a paper overlay on top of white membrane. Snap cover back on.

**To recharge battery:** Battery will run for up to 15 hours of continuous use. When battery is low, an orange light will flash towards the end of message playbacks. A micro USB cord is provided for recharging.

*NOTE: Routinely recharging the unit is recommended to enhance battery life and usage.*



**Level Lock:** Press and hold the green **Level button** and the **Volume Up button** simultaneously until a tone is heard. This will lock you into the level you are currently on.

**To Undo Level Lock:** Press and hold the green **Level button** and the **Volume Down button** simultaneously until a tone is heard. You will now be able to change levels.

**Record Lock:** Press and hold the red **Record button** and the **Volume Up button** simultaneously until a tone

is heard. The record feature is now disabled.

### **To Undo Record Lock:**

Press and hold the red **Record button** and the **Volume Down button** simultaneously until a tone is heard. The record feature is now enabled.

